

BB SKIN



ULTRASONIC FOR FACE & FOR BODY 2in1

SKIN1 USER MANUAL

PRODUCT DESCRIPTION (1)

■List of product parts:

Mainframe × 1

Ultrasonic tip for body × 1

Targeted ultrasonic operational handle × 1

Handle head × 3

Handle stand × 1

Power adapter × 1

User manual × 1

Warranty & Certificate × 1



Mainframe and Mainframe Interface



Ultrasonic tip for body



Targeted ultrasonic handle and operating tip
(standard device includes one handle and three operating tip)



Handle stand



Power adapter

PRODUCT DESCRIPTION (2)

■ Ultrasonic tip for body



The round button next to the handle seam is the Ready/Standby button: blue light means standing-by; red light means ready.


The long bar button located on the Operating Head is for power on and off: the circle on top is for power off, while the liner bar is the button for power starting; blue light and red light alternating flash when power is importing, while the blue light stays on when power stops (power import regularly when the red button appears in red).

There is superconducting liquid inside the equipment tip. Do not shake the tip.

Power light	The blue light and red light alternating flash when power is importing, while the blue light stays on when power stops.
Power importing button	Press down to import power.
Power adjustment button	Increase or decrease power.
Ready/Standby button	Red light means the equipment is ready to use, while blue light means it's on standby.


PRODUCT DESCRIPTION (3)

Targeted ultrasonic handle

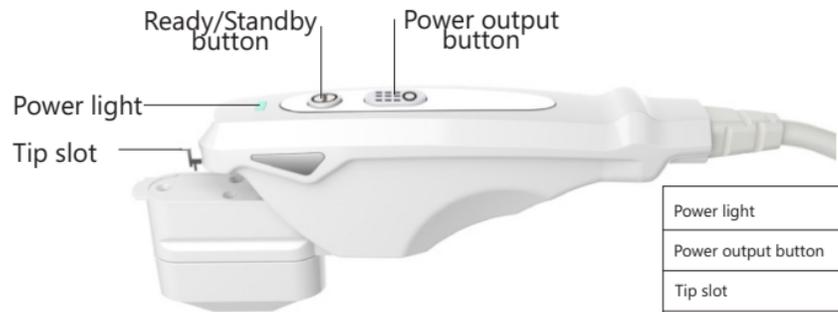

Power light	Red light flashes and beeps when power is exporting.
Power output button	Press down for power output
Tip slot	To block tip
Ready/Standby Button	Ready to use when red light is on, standby when blue light is on.

Operating tip

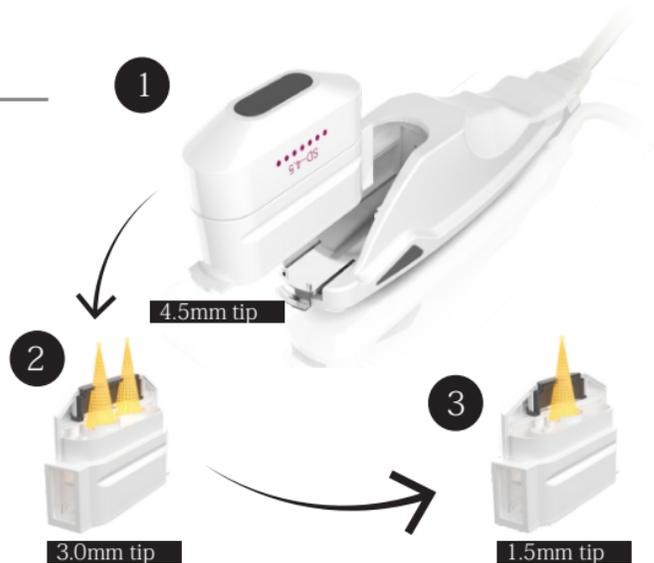

Special notes:

Targeted ultrasonic applies to tips of different lengths: 4.5mm, 3.0mm and 1.5mm.

Different lengths work on different depth. Using tips of different lengths for the same area would in turn go to SMAS, subcutaneous fat and Dermis, creating the effect of tightening, lifting and stimulating collagen.

Suggestion: use 4.5mm tip for neck, chin and lower face; use 3.00mm tip for apple cheek; use 1.5mm tip for forehead and eye circles, as well as the whole face area.

Could also start with 4.5mm tip, remove the tip and switch to the 3.0mm tip and so forth.

There is superconducting liquid inside the equipment tip. Do not shake the tip!

The function of this product include Targeted Ultrasonic for both body and face .
Ensure proper connection and use of different unit before application (see image below).



Please ensure respective red dots and numbers match when connects to the mainframe.



Ultrasonic for body



Ultrasonic for face

ULTRASONIC FOR BODY Operation interface

Depth: 8mm, 11mm, 15mm

8 mm, 11 mm, 15 mm

30:00 TIME (MINUTES)

0.5 FLUENCE(JOULES/CM²)

MODE A, MODE B, MODE C, **STANDBY**, HOME

⑥ Depth indicator ① Operation mode ④ Standby/Ready button

② Operating time ③ Power adjustment ⑤ Back to home page button

ULTRASONIC FOR FACE Operation interface

Depth: 1.5mm, 3.0mm, 4.5mm

1.5 mm

THE NUMBER OF [15000 \ 15000] DS - 1.5

TIME ENERGY POINTS

30:00 3.0J 3

STANDBY

HOME

① Number of remaining use for the tip ② Operating time ③ Power adjustment

④ Output times ⑤ Operating areas ⑥ Type pf operating tips ⑦ Standby/Ready button ⑧ Back to main page button

ULTRASONIC FOR BODY APPLICATION EFFECTS



Reduce body fat for different parts of body and all skin types.

Effectively shape body, reduce tissue sagging and cellulite for localized areas.

Body shaping for arms and legs, abdomen, chest and back areas.

Tightening and lifting: Recover body shape for post-pregnancy and liposuction.

Weight-loss: reduce body fat, improve sagging skin.

Skin tightening: improve skin elasticity, improve the appearance of cellulite and stretch-marks caused by pregnancy.

ULTRASONIC FOR BODY

Ultrasonic for Body Detailed Operation Manual




ULTRASONIC FOR BODY OPERATION MANUAL (1)

Place the unit of Ultrasonic for Body on the equipment, then connect to the power source.


Turn on the power button and start the equipment.

Click to open the operation interface.

ULTRASONIC FOR FACE INTERFACE

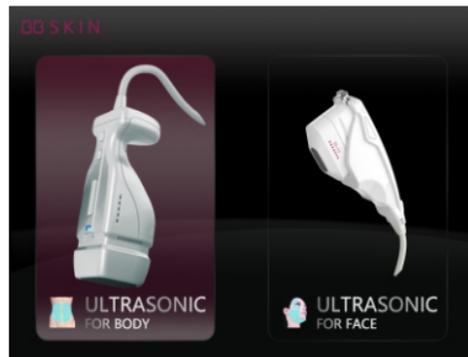

ULTRASONIC FOR BODY Operation interface

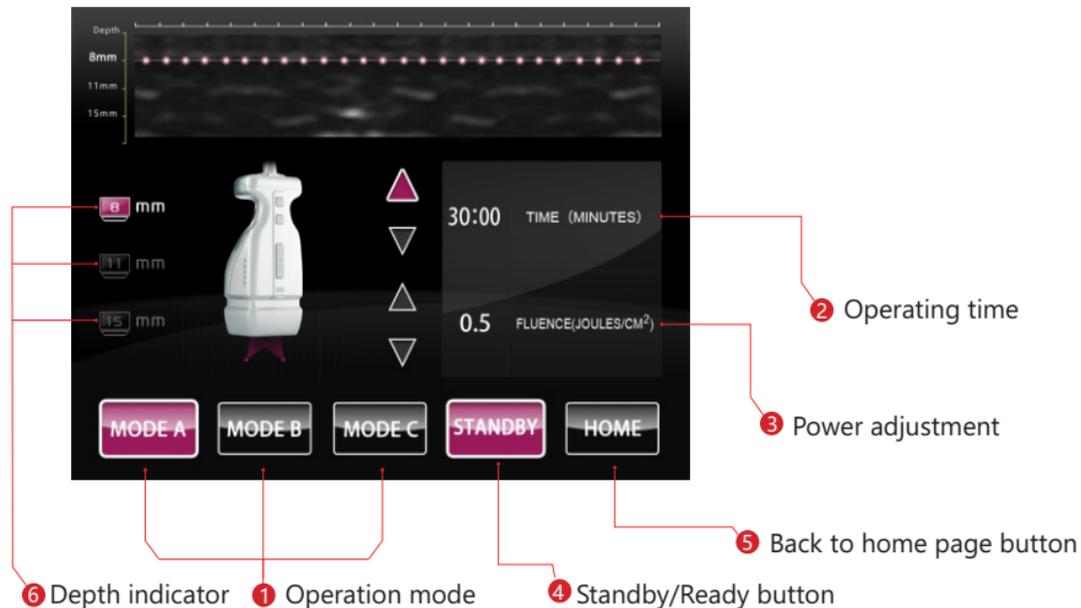

ULTRASONIC FOR BODY OPERATION MANUAL (2)

1. Operation mode:

1) Mode A: for people who have loose skin due to pregnancy, aging, weight loss, etc.

Treatment frequency: 30 – 45 minutes per treatment every 3 days. 10 treatments complete 1 session.

Operation: power starts from 1.0 J and gradually increases to 2.0 to 3.0 J, when a tingling sensation on the skin starts; operate tightly on contact of skin and continuously move the operating unit tip on targeted area; make sure gel distributes evenly on target area and on the tip and re-apply gel when necessary. Do not operate on skin without gel.

2) Mode B: for people who are slightly overweight with troubling areas, including saggy buttock, cellulite on thighs, upper arms, back and the abdominal area.

Treatment frequency: 30 – 60 minutes per treatment every 4 days. 10 treatments complete 1 session.

Operation: power starts from 2.0 J and gradually increases to 2.5 to 3.5 J, when a tingling sensation on the skin starts; operate tightly on contact of skin and

continuously move the operating unit tip on targeted area; make sure gel distributes evenly on target area and on the tip and re-apply gel when necessary. Do not operate on skin without gel.

3) Mode C: for people who are overweight.

Treatment frequency: 30 minutes per treatment every 5 days. 10 treatments complete 1 session.

Operation: power starts from 3.0 J and gradually increases to 2.5 to 3.5 J, when a tingling sensation on the skin starts; operate tightly on contact of skin and continuously move the operating unit tip on targeted area; make sure gel distributes evenly on target area and on the tip and re-apply gel when necessary. Do not operate on skin without gel.


Note: Due to various level of sensitivity and tolerance on individuals, the power setting of the above mentioned three modes shall be adjusted base on individual situation; it shall be started from 0.5 J and increases to a level under individual tolerance. Treatments can be more frequent as treatment times increase.

ULTRASONIC FOR BODY OPERATION MANUAL (3)

2. Operating time: operating time can be set on the right hand side of the screen using up and down arrows; "Mode A" lasts 90 minutes max, "Mode B" lasts 60 minutes max. and "Mode C" lasts 30 minutes max.

3. Intensity adjustment applies only when the power is not exporting: power can be adjusted by the up and down arrow keys Fluence (joules/cm²) at the right hand side of screen and usually 2 to 3 J (power can also be adjusted by the "+" and "-" buttons on the handle; power changes every time when the "+" and "-" button is pressed). It is suggested that adjustment should refer to skin tolerance at a comfort level.

4. During operating process, the tip should keep close contact on skin and **ensure the gel is evenly and generously spread. Make sure the tip moves continuously when power starts exporting on targeted area. Do not stay on one area over 3 seconds.** Note: the intervals between contacts should not be too long, in order to ensure power applies to treated skin evenly.

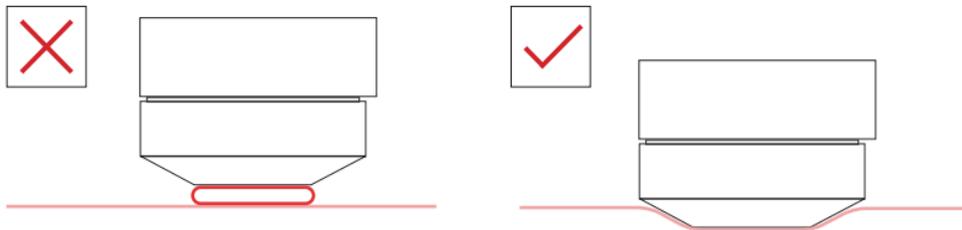

5. Avoid operating on belly button, rib area, or damaged skin; do not operate during menstrual period or on individuals who have metal implants in their body.



Special note: navel and ribs operation is prohibited.

6. Prior to treatment, remove all metal accessories on body and clean treatment area. Apply specialized gel on skin and start treatment based on the following instructions:

6.1. Starting the "STANDBY" mode: apply specialized gel to the operating tip and get contact on skin, press "STANDBY" on the screen (also could press the "Ready/Standby" button on the handheld operating device). Once "STANDBY" mode starts, the LED light on the handheld device turns in red (when the LED light turns in blue, the device is in idle mode; the device will not have output power when pressing the output button).

6.2. Output power operation: place tip closely on skin, press the "Start outputting" button; when the LED light starts to flash in blue and red, it means the tip is in outputting power; pressing the "Stop outputting" button, the LED light turns in blue and the power stops outputting.

ULTRASONIC FOR BODY OPERATION MANUAL (4)

7. Operation method (**special instruction: apply generous amount of gel on tip before use. When the tip starts power outputting, it must continuously move on the targeting area. Do not stay on one area for over 3 seconds.**)

Abdominal area: glide on skin using horizontal directions and then vertical. One area can be operated between 3 to 5 times. Avoid the bellybutton and rib areas.



Arms: move in circular motion on one area and move to another area horizontally.



Thigh and buttock: move in circular motion on one area and move up to another area. Finish with one line and start another. Then repeat horizontally until the operation covers the entire thigh or buttock area.



8. Once the operation is completed:
Clean the gel and apply body lotion (once a day).
Keep body hydrated. Drink at least 2000ml of water.
If treatment is for stretchmark reduction, use stretch-mark treatment cream.

ULTRASONIC FOR BODY RECOMMENDED TREATMENT FREQUENCY

Treatment is recommended by sessions; each session is completed by 10 treatments.

1st Session: 3 to 5 days between treatments;

2nd Session: 7 days between treatments;

3rd Session and later: 15 days between treatments.

*For overweight patients, treatments can be more frequent based on individual tolerance.

ULTRASONIC FOR BODY PRECAUTIONS (1)

1. Before use:

- 1.1** Do not use on people who are in pregnancy or with major disease (heart disease, high blood pressure, hyperthyroidism, asthma, cancer or mass bleeding).
- 1.2** Do not use on people who have skin disease, open wound or acne.
- 1.3** Do not operate while the patient is under treatment for any disease, unless permitted by a licensed doctor.
- 1.4** Not recommended to be used during menstrual period.
- 1.5** Do not operate on skin without special gel.
- 1.6** Make sure to clean and dry treatment area before the operation and remove all metal accessories from body.
- 1.7** People who have injected with hyaluronic acid must wait for 15 days to receive treatment.
- 1.8** Not recommended for people who are allergic to ultrasonic or plastics.
- 1.9** Do not use on people who had organ implant containing metal.
- 1.10** Do not operate on rib areas (although it hasn't been evidently proven that ultrasonic may cause osteoporosis, it is still not recommended on the rib areas).
- 1.11** Do not use during the first three months after giving birth.

2. During use:

- 2.1** Make sure not to stay on one area for more than 3 seconds. The tip needs to be moved continuously on treatment area.
- 2.2** Do not press the power output button without gel applied; otherwise it may reduce the life of the machine.
- 2.3** Do not use on areas that have received any injections or surgery.
- 2.4** Make sure 3mm of gel is applied on both treatment skin area and the tip; keep checking during operation and reapply the gel if needed.
- 2.5** Power is to be applied from low to high to a tolerable level (slight tingling sensation on treatment area) and keep tip pressed evenly on skin).
- 2.6** There is superconducting liquid inside of the tip. Do not shake it.

3. After use:

- 3.1** Protect treatment areas from sun and heat. It is normal that treatment areas experience slight redness, rash or swelling after the operation. Usually it will disappear after 1 or 2 days.
- 3.2** Keep body hydrated. Do not take warm bath on the day of treatment; do not have spa or massage within 2 weeks after treatment; avoid spicy food, alcohol and smoking; eat more fruits and vegetables.

ULTRASONIC FOR BODY PRECAUTIONS (2)

3.3 Skin redness:

Cause: the power goes deep under the skin and stimulates blood circulation.

Solution: keep treated area hydrated and well-moisturized; normally the redness will disappear after 1 or 2 hours.

3.4 Fat granule or acne few days post-treatment:

Cause: skin was not thoroughly cleaned before treatment; the treatment has high level of targeted power input, which may lead sebum, dirt or makeup into the skin and clog the pores. Targeted ultrasonic also has detox effect; if the skin has significant amount of toxins, the treatment will help eliminating the toxins from the skin, which may cause fat granule or acne.

Solution: apply Vitamin B6 cream and anti-allergy pill (for external use) on affected area. Crush the pill and mix with VB6 cream and apply on the redness area. If only experiencing acne without redness, applying VB6 cream alone is recommended, 3 to 5 times per day.

3.5 Skin is dry and itchy

Cause: after operation, will lost more moisture, hydrating not enough or not in time, or insolate, or at the computer for a long time, can lead to dry, itchy skin.

Solution: within a week, every day hydrating, Do not insolate, Avoid long time computer, TV and so on high power display of radiation.

3.6 Skin shows redness, swelling and flaky

Cause: It usually happens on sensitive skin. Aging, stress, air pollution, cosmetics, cold or heat and invasive cosmetics procedures (injection, implants or plastic surgery) will all likely cause facial sensitivity and thin surface layer. When ultrasonic power gets directed into skin, it may cause localized redness, swelling and flakes.

Solution: keep body hydrated. Usually the redness will disappear within a week and Vitamin C intake will help with the recovery process.

3.7 Skin shows bruises and yellow

Cause: usually skins that had cosmetic procedures or injections may show bruises or yellow after Ultrasonic for Body treatment. Those procedures cause thin and week blood vessels that are easily breakable. After targeted ultrasonic treatment, the thin blood vessels may break and the skin will show bruises and yellow due to localized blood congestion.

Solution: normally the bruise will disappear after a week, or apply medical creams that can help with reducing the appearance of bruises.

3.8 Effect is not obvious

Cause: Ultrasonic for Body is a physical fat burning procedure. It destructs the aged cell tissue and stimulates skin's collagen production system to accelerate self-recovery. For people over 30 years old, the skin's ability for collagen production

ULTRASONIC FOR BODY PRECAUTIONS (3)

and Hyaluronic acid level reduce quickly, which will cause dehydrated skin and nutritional imbalance.

Solution: collagen intake to boost the body's collagen level that will improve the effects.

4. Prevention of skin blistering

4.1 Continuously apply skin hydration product once every day, 15 minutes each time, starting 1 week prior to treatment; use warm water applications on abdominal area 1 to twice every day, 15 minutes each.

4.2 For people with sensitive skin, suggesting to use warm water application on targeted area 1 to twice every day, 15 minutes each, starting 1 week prior to treatment, so as to let skin get used to heat stimulation (water should not be too warm. If skin turns red immediately when gets on contact of heat after one week, please postpone treatment).

4.3 Apply warm water on treatment area for 15 minutes prior to treatment.

4.4 Power setting should follow week to strong, short to long treatment time and Mode a-b-c.

4.5 Press firmly onto skin and make sure tip is closely on contact with treatment area.

4.6 If skin tingling sensation is strong or the temperature is too high, the tip should

stop contacting skin.

4.7 Make sure tip is closely pressed on skin before turning on the power button; if operating on thin fat area, it should be squeezed and make sure the tip is closely pressed on before starting power output.

5. Actions for post-treatment swelling blisters

5.1 If swelling blister occurs, apply ice area for 10 minutes; If swelling doesn't disappear, apply ice treatment within 24 hours.


5.2 If skin appears to have small blisters, apply a thin layer of burning treatment cream and do not get on contact with water within 24 hours.

5.3 If skin appears to have large blisters, use Iodophor to disinfect the area and use 1ml injector to take out the fluid, disinfect with Iodophor again and apply a thin layer of burning treatment cream. Do not get in contact with water within 24 hours.


5.4 If blisters continue to occur or come back, use the same method mentioned above; when the dead skin layer on the blister peels off, apply Vitamin B cream and Vitamin C powder (small amount) on affected area and protect from sun exposure. Stop the application until skin appears to normal colour.

ULTRASONIC FOR BODY TECHNOLOGY PRINCIPLES


Through targeted ultrasonic technology, ULTRASONIC FOR BODY operates on different depth of skin layers and use heat to destroy under skin fat permanently through thermal power, stimulate collagen production, fill in the gap after fat reduction and smooth out skin surface and tighten skin tissue.

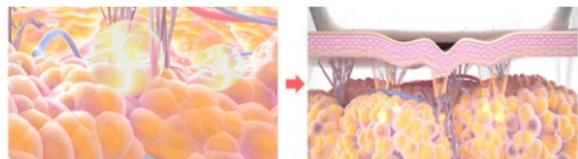

Operate on under skin fat cells, destroy and eliminate the layers of fat.

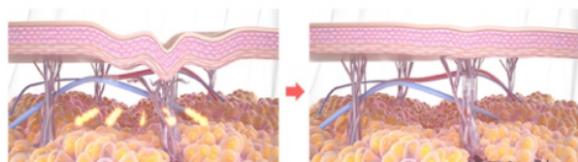


Stimulate collagen production under skin and tighten the skin on the treated area.

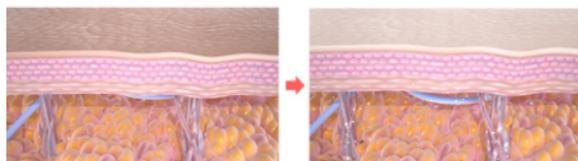


The dual action can effectively reduce fat under the targeted area without any invasive procedures to achieve toning and tightening effect.


OPERATING PROCESS

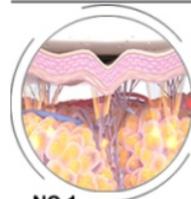

NO.1
Place the tip on the treatment area where gel is applied.

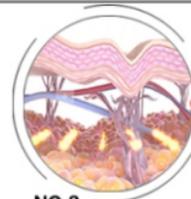

NO.2
The device will export high-frequency targeted ultrasonic power to destroy the fat cells under skin layer.

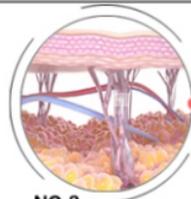

NO.3
Power can go under the skin layers without damaging the skin surface.



NO.4
Destroy fat cells will be excreted from the body through the metabolism process.



NO.5
stimulate collagen production, Eliminate wrinkles and stretch marks on abdominal area.

DEVICE ADVANTAGE



Melting and reducing fat

Targeted ultrasonic power destroys fat permanently and reduces the amount of fat cells.



Intelligent system

Intelligent scanning system targets fat layers under skin without damaging the skin surface.



Safe and non-invasive procedure

Produce heat to melt fat without any invasive surgical procedures.



Significant effect without rebound

Permanently damages fat layers under skin, reduces the number of fat cells without rebound effect.



Tightening and smoothing

While steatolysis, stimulate collagen regeneration. Fill the subcutaneous fat. The treated skin appears smoother and more supple.

ULTRASONIC FOR FACE EFFECTS



Mostly to be used for the improvement of loss of elasticity of facial skin, wrinkles and uneven surface. Can help shaping facial contour, lifting and tightening skins surrounding the eye and neck areas.


1. Anti-aging and wrinkle improvement: reduce wrinkles on the forehead, nasolabial folds, between the eye brows, on and around the lip areas.
2. Wrinkle improvement around eye area: eliminating crow's feet, tightening and lifting loose skin;
3. Wrinkle improvement on neck, eliminate double chin effect, preventing aging neck skin.
4. Improve overall skin quality – smooth surface and increase elasticity, eliminate excessive facial fat, tightening for better-looking facial contour.

ULTRASONIC FOR FACE


DETAILED OPERATION MANUAL



ULTRASONIC FOR FACE OPERATION MANUAL (1)

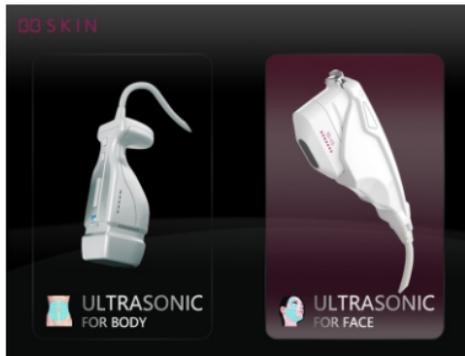


Plug the Ultrasonic for Face tip to the device and connect to the power cord. Turn on the power, start the machine. Click into Ultrasonic for Face operation interface.

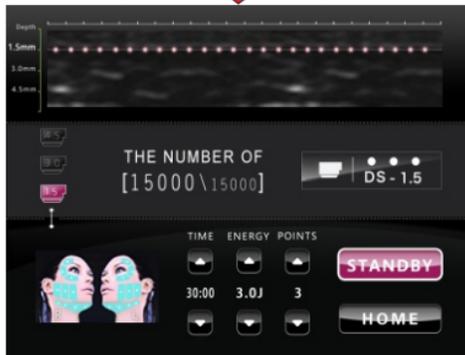

ULTRASONIC FOR FACE OPERATION INTERFACE.

ULTRASONIC FOR FACE Operation interface

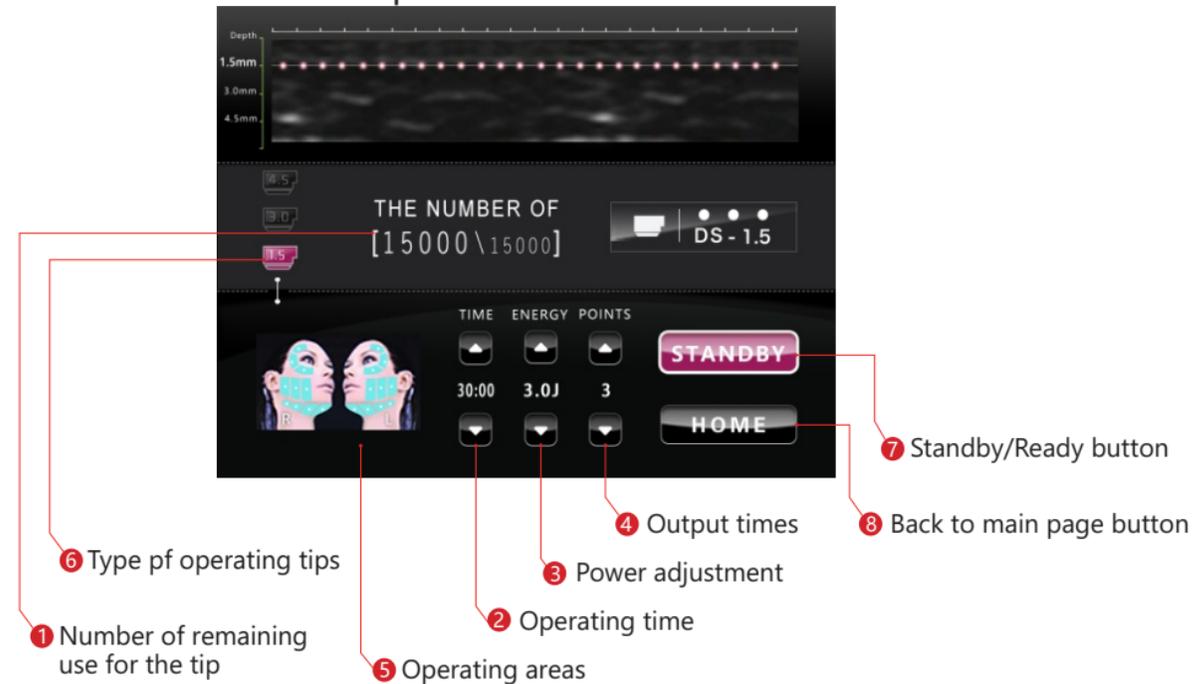

ULTRASONIC FOR FACE OPERATION MANUAL (2)

1. Power adjustment:

Start with 0.5J/Point and 3 points/time, then gradually increase power as needed under tolerance; as power increases, the tingling or warm sensation is normal. Approximately 20 minutes of operation, the skin will become red; it is completely normal due to the skin collagen's thermal reaction on the surface.

Everyone's skin has different tolerance level and therefore the selected power level can be different. Always start from low power and gradually adjust to the most tolerate power level. Do not start from the strongest otherwise it might cause skin damage.




Time: 1 minute to 60 minutes

Energy (power level): 0.5J to 5J

Points (output points/time): 3 points/time to 25 points/time

2. During operation, the tip should be closely pressed onto skin without gaps and make sure gel is evenly and generously applied.

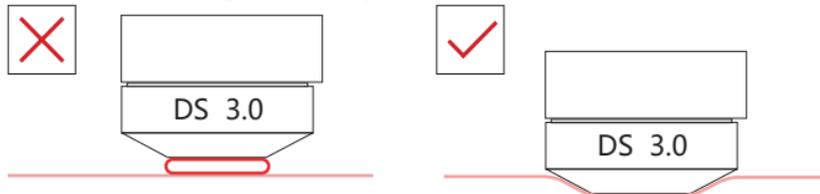


3. Do not operate around the eye area, or on damaged, acne prone skin. Do not operate on any areas that have received implants. Collagen or hyaluronic acid injected area should wait for a month before operation.



Special note: Do not operate on 1 cm radius around the eye frame, prominentia laryngea or temple area.

4. When operating on neck and cheeks, suggesting to adjust the power level to 1.5-2.0J\Points 3; when operating on the forehead, suggesting to adjust the power to 0.5-1.0J\Points 3

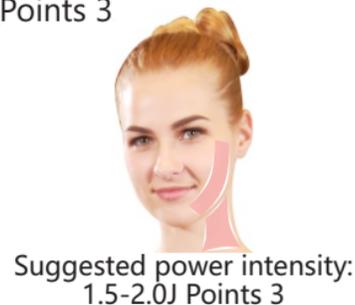

ULTRASONIC FOR FACE OPERATION MANUAL (3)

5. Operation techniques:

The machine will start beeping when the tip starts outputting power. Individuals can choose the following methods based on experience.

Method A:

Fixed point - suitable for inexperienced individuals. When the tip starts working, do not move the tip until the power output light stops flashing. **This method requires not exceeding 15 points/time and the power level can be set as 3.0J - 3.5J according to individual tolerance. If the skin experiences discomfort during operation, move the tip away from skin immediately.**

Method B:

Moving points - suitable for experienced individuals. Continuously move the tip on treatment area and when finishing each area, press the power output button again. Operating time for each treatment area should be no longer than 5 minutes. This method requires points to be set at 25 points/time (to reduce the times to press the button) and the power is to be set at 3.5J - 4.0J according to individual tolerance level. **If the skin experiences discomfort during operation, move the tip away from skin immediately.** Before operation, the treatment area should be thoroughly cleaned and all metal accessories should be removed.

Apply specialized gel on treatment area, place the proper tip on the device, adjust the power from low to high based on skin tolerance and operate by the following procedures (press the tip firmly onto skin; the gap between operation time should be closeness):



Neck:

Operating tip starts from the bottom of the chin and move up to the ear tip. Do not operate on prominentia laryngea.



Face:

Operating tip starts from the corner of the mouth and moves up on a curved line towards the ear tip.



Cheekbone:

Operating tip moves on a fixed area under the cheekbone on a vertical curved line.



Surrounding eye area:

Operating tip moves on a fixed area from the bottom of the eye up to the corner on a curved line. Do not operate on upper eyelid and eyeballs.



Forehead:

Operating tip moves on a fixed area from the bottom to the upper forehead area.

6. When the treatment is finished:

Clean face thoroughly and apply facial serum and mask. Remove the mask after 10 minutes. If the mask gets dry, re-apply water to the mask.

Avoid highly-heated environment. Redness, rash, minor swelling or blisters are normal reactions, which will disappear within a day or two.

Try to have more skin hydration treatment. Do not use makeups within 3 days after treatment. Do not have hot bath on the day after treatment. Do not have hot sauna or facial massage within 2 weeks after treatment.

Avoid spicy food, smoking and alcohol. Try to eat more fruits and vegetables and have healthy diet.

ULTRASONIC FOR FACE RECOMMENDED TREATMENT FREQUENCY

Treatment is recommended by sessions; each session is completed by 10 treatments.

1st Session: 7 to 10 days between treatments;

2nd Session: 10 – 15 days between treatments;

3rd Session and later: 20 days between treatments.

*For sensitive skin, 10 days between the first three treatments; after the 4th treatment, it could be two weeks or one month apart from each treatment. Adjust on individual cases.

ULTRASONIC FOR FACEPRECAUTIONS (1)

1. Before use:

- 1.1 Do not use on people who are in pregnancy or with major disease (heart disease, high blood pressure, hyperthyroidism, asthma, cancer or mass bleeding).
- 1.2 Do not use on people who have skin disease, open wound or acne.
- 1.3 Do not operate while the patient is under treatment for any disease, unless permitted by a licensed doctor.
- 1.4 Not recommended to be used during menstrual period.
- 1.5 Do not operate on skin without special gel.
- 1.6 Make sure to clean and dry treatment area before the operation and remove all metal accessories from body.
- 1.7 People who have injected with hyaluronic acid must wait for 1 month to receive treatment.
- 1.8 Not recommended for people who are allergic to ultrasonic or plastics.

2. During use:

- 2.1 Make sure not to stay on one area longer than 10 seconds, when the power output sets at 4.0J+. When the skin feels any discomforts (including overheat or strong tingling sensation), stop the operation immediately. The tip needs to be moved continuously on treatment area.
- 2.2 Do not press the power output button without gel applied; otherwise it may reduce the life of the machine.

- 2.3 Do not use on areas that have received any injections, implants or surgery.
- 2.4 Do not operate surrounding the eye, prominentia laryngea and temple area, especially avoid move the tip over eye ball.
- 2.5 Make sure 3mm of gel is applied on both treatment area and the tip; keep checking during operation and re-apply the gel if needed.
- 2.6 Power is to be applied from low to high to a tolerable level (slight tingling sensation on treatment area) and keep the tip pressed evenly on skin).
- 2.7 There is superconducting liquid inside of the tip. Do not shake it!

3. After use

- 3.1 Protect treatment areas from sun and heat. It is normal that treatment areas experience slight redness, rash or swelling after the operation. Usually it will disappear after 1 or 2 days.
- 3.2 Keep body hydrated. Do not wash face with warm water on the day after treatment is received. Do not use makeups within 3 days after treatment. Do not have hot bath on the day after treatment. Do not have hot sauna or facial massage within 2 weeks after treatment.
Avoid spicy food, smoking and alcohol. Try to eat more fruits and vegetables and have healthy diet.
- 3.3 Skin redness:
Cause: the power goes deep under the skin and stimulates blood circulation.

ULTRASONIC FOR FACEPRECAUTIONS (2)

Solution: keep treated area hydrated and well-moisturized; normally the redness will disappear after 1 or 2 hours.

3.4 Fat granule or acne few days post-treatment:

Cause: skin was not thoroughly cleaned before treatment; the treatment has high level of targeted power input, which may lead sebum, dirt or makeup into the skin and clog the pores. Targeted ultrasonic also has detox effect; if the skin has significant amount of toxins, the treatment will help eliminating the toxins from the skin, which may cause fat granule or acne.

Solution: apply Vitamin B6 cream and anti-allergy pill (for external use) on affected area. Crush the pill and mix with VB6 cream and apply on the redness area. If only experiencing acne without redness, applying VB6 cream alone is recommended, 3 to 5 times per day.

3.5 Skin is dry and itchy

Cause: after the targeted ultrasonic treatment, skin will lose its moisture. If hydration is not adequately supplied or the treated skin is under sun exposure or facing computer for extended period of time, the skin will become dry and itchy.

Solution: apply hydration mask daily in the week after treatment. Avoid exposure under the sun or long time facing the TV or computer.

3.6 Skin shows redness, swelling and flaky

Cause: It usually happens on sensitive skin. Aging, stress, air pollution, cosmetics, cold or heat and invasive cosmetics procedures (injection, implants or plastic

surgery) will all likely cause facial sensitivity and thin surface layer. When ultrasonic power gets directed into skin, it may cause localized redness, swelling and flakes.

Solution: keep body hydrated. Usually the redness will disappear within a week and Vitamin C intake will help with the recovery process.

3.7 Skin shows bruises and yellow

Cause: usually skins that had cosmetic procedures or injections may show bruises or yellow after Ultrasonic for Body treatment. Those procedures cause thin and week blood vessels that are easily breakable. After targeted ultrasonic treatment, the thin blood vessels may break and the skin will show bruises and yellow due to localized blood congestion.

Solution: normally the bruise will disappear after a week, or apply medical creams that can help with reducing the appearance of bruises.

3.8 Effect is not obvious and fine lines appear

Cause: Ultrasonic for Body is a physical fat burning procedure. It destructs the aged cell tissue and stimulates skin's collagen production system to accelerate self-recovery. For people over 30 years old, the skin's ability for collagen production and hyaluronic acid level reduces quickly, which will cause dehydrated skin and nutritional imbalance leading to fine lines on face.

Solution: collagen intake to boost the body's collagen level that will improve the effects.

TARGETED ULTRASONIC TECHNOLOGY PRINCIPLES

Targeted ultrasonic technology has been applied in the medical field for over 50 years. It is proven to be safe and effective. In the early stage of medical use, it was adopted to eliminate diseased tissues, such as tumor; later it was discovered that this technology had significant effectiveness when used for skin improvement and fat reduction, therefore it has been widely used in the medical aesthetics.


Home-use Targeted Ultrasonic is based on High Intensity Focused Ultrasound, which targets ultrasound on single spots, generates heat and applies on SMAS (Superficial Musculo-Aponeurotic System). It shrinks the SMAS layer of the skin and further stimulates collagen renewal and production in the deep layers. Meanwhile, as the power skims over the skin surface, it will not damage the skin but tightens and smooths.



WHAT IS SMAS?

If we see our skin as a house, SMAS is the foundation that supports it. If the foundation is unstable, the house will collapse. Likewise, if the SMAS layer is damaged and losing the power of support, wrinkles and sagging skin will happen.


As the foundation support of the face, strong SMAS plays a key role in firming skin. Targeted Ultrasonic is the only non-invasive procedure that helps strengthen the SMAS layer, in order to improve facial contour and youthful appearance.

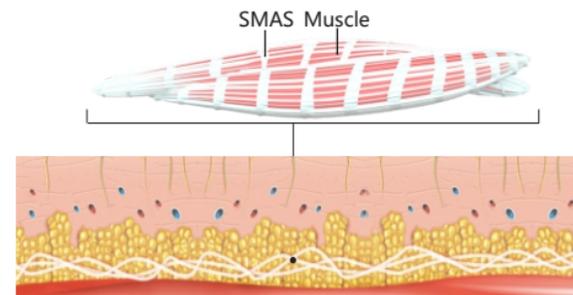

TARGETED ULTRASONIC TECHNOLOGY PRINCIPLES


OPERATION DEPTH AND TEMPERATURE

Operation temperature

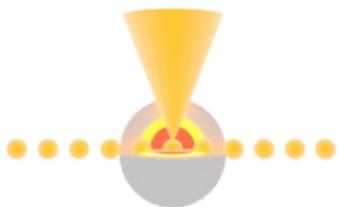

When Targeted Ultrasonic operates on the deeper layers of skin, it will cause thermal contraction that immediately increases the skin cell temperature to 60-70 degrees Celsius, which is the ideal temperature for collagen renewal and production.

Operation depth

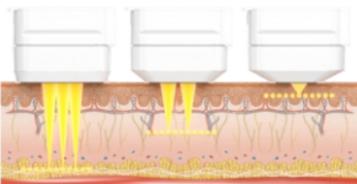

Targeted Ultrasonic has 3 different tips (4.5 mm, 3.0 mm and 1.5mm) that could go to different layers of skin - SMAS, subcutaneous fat and dermis, subsequently could create effect of skin firming and collagen reproduction.

DEVICE ADVANTAGE



Optimal temperature

When Targeted Ultrasonic operates on the deeper layers of skin, it will cause thermal contraction that immediately increases the skin cell temperature to 60-70 degrees Celsius, which is the ideal temperature for collagen renewal and production.




Precise localization

With the most precise, patent-protected "Localization Indicator Line", the device is able to work on spots that precisely directs power to the skin. The manually controllable power level adds more safety measure.



Precise depth

The power targets at different depths of skin tissues and has no damage the the surface skin, which works deeper layers for better effects as well as protecting the skin surface.

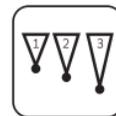

Multiple tips

3 different tips work on different depths: 4.5mm, 3.0mm and 1.5mm. Select different tip based on skin type and targeted depth. The device can be operated on various areas, including face and neck for great results.

DEVICE MAINTENANCE

1. Please clean the tip with damp cotton-made cloth after each use and properly store it.
2. Prior to use, make sure to use proper adapter for the machine and power is properly plugged in.
3. Make sure the power voltage is compatible. If the local power voltage is unstable, suggesting users to increase stable power source with compatible voltage.
4. To ensure machine's regular performance life, please consistently use parts provided or recommended by the manufacturer.
5. Do not place the machine close to humid places or next to the water; also do not place it under direct sun exposure.
6. Do not place the machine close to a heat source, which might affect the machine's life cycle and normal performance.
7. Use current circuit sleeves in winter or when the weather is dry. In order to prevent poor contact, please make sure the skin-contacting area of the tip is damp. Use any water-based product or simply water to wipe the tip area is recommended.
8. Water, oil, etc should not enter inside the machine. To prevent damage, do not strike or beat the machine.
9. Clean the tip and cut off power after use.
10. The machine comes with a 1-year warranty. As machine still keeps performing, regular maintenance and parts changing are free under the warranty.
11. Life-long maintenance, repair and technical support are available.
12. Do not clean the machine with products containing alcohol.
13. There is superconducting liquid in the operation tip. Do not shake the tip.

